TOMYDOO

M O D E D ' E M P L O I

DATE DE VENTE :

GARANTIE 1 AN

CACHET DU REVENDEUR • CARIMBO DO DISTRIBUIDOR
STEMPEL DES HÄNDLERS • RETAILER STAMP

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
- 1 Main unit
- 2 Silicone end nozzle
- 3 Cup
- 4 Cup base
- 5 Silicone join
- 6 Secretion evacuation space
- 7 Fastening base
- 8 Suction hole
- 9 Draining hole
- 10 Switch
- 11 Battery compartment
- 12 Batteries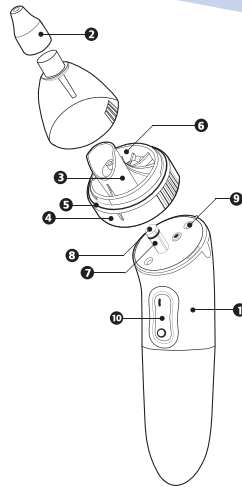

RECOMMENDATIONS

- To improve the efficiency of the appliance, it is recommended to block one nostril while using it on the other.
- A few seconds are enough to remove secretions. Do not use the nose cleaner in the case of nasal irritation.
- It is recommended to clean the nozzle part between each use.
- Store the product in a dry place at room temperature. If not used for several months, it is recommended to remove the batteries.
- The Tomydo nose cleaner helps prevent respiratory complications. In the case of infection, please consult a doctor.


INSTRUCTIONS FOR USE

Inserting the batteries

open the battery compartment (11) by removing it from the nose cleaner (1). Insert two AA LR6 batteries then shut the compartment.

Switching on

press the upper part of the switch (10) and the lower part to stop the vacuum.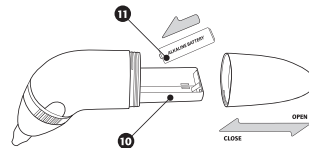

Using the appliance

position the silicone nozzle (2) just at the opening of the obstructed nostril for a few seconds. The secretions are collected in the secretion space (6).


DELIVERED WITH 3 SILICONE END NOZZLES

- white for thick nasal secretions.
- blue for regular nasal secretions
- pink for new born baby between 0 to 3 months

CLEANING AFTER USE


Removing the nozzle part

- Pull the silicone nozzle (2).
- Pull the upper part including the cup base (4) to remove it from the rest of the main unit of the nose cleaner (1).
- Unscrew the cup (3) for cleaning
- Make sure that the silicone joint is in the right position (5).
- Screw back and reassemble the different parts.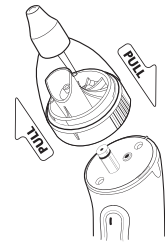


Cleaning the Nozzle part

remove the nozzle part from the nose cleaner (2, 3, 4, 5, 6, 7) and rinse the different parts with water. They can be sterilised with boiling water or steam.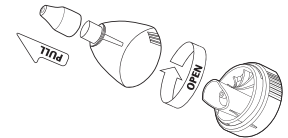

Cleaning the Main unit

make sure that the battery compartment is shut. After having removed the nozzle part, immerse the suction hole (8) of the appliance in water and switch on. In a few seconds the appliance is clean. Do not put the switch in water.

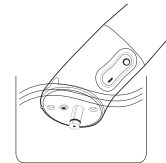PRECAUTIONS

Tomydo is designed to clear babies' nasal secretions.

IMPORTANT: *Tomydo should not be used in any conditions other than those described above. The battery compartment SHOULD NOT be immersed in water.*

- Before use, make sure that all the different parts are correctly assembled.
- Do not push the nozzle into the baby's nostril.
- Keep Tomydo away from children's reach.
- Always close the battery compartment when cleaning the appliance.
- Do not undo the screws or joints. This could damage the appliance.
- Do not clean Tomydo with solvents or abrasive products.

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